

# True-Sense Kit, ReLax Science Kit & ReLax Kit

## Instructions for Charging, Turn-on and Turn-off


Our products are partially charged ex-factory with both Controller and Sensor in "OFF" mode for storage. Please charge the Kit before using for the first time. The procedures are as follows:

1. Plug Sensor into Controller so both can be charged. (If the Kit is already over-discharged, the charger will not start charging both together due to high current protection, please charge Controller individually first, then charge up the Sensor.)
2. Connect Controller to any USB charger (or PC\*), the orange LED on Controller should light up. If the orange LED flashes, it indicates the batteries are over-drained and Controller is not detectible by the PC. Please keep the Kit connected to the charger, in few minutes, the orange LED will turn to steady on, then normal charging will start.
3. The Kit should be fully charged in 30 minute time.
4. After the foregoing step, Sensor and Controller will still be in "OFF" mode. Please remove the Kit from the charger.

There are two buttons on Controller, the "+" is the ON button and the "-" is the OFF button.

With Sensor plugged into Controller, press the ON button for more than 2 seconds, all Controller LEDs will flash once, both Sensor and Controller are turned on; remove Sensor from Controller. Now you are ready to use the Kit.

With Sensor plugged to Controller, press the OFF button for more than 2 seconds, all Controller LEDs will flash once, both Sensor and Controller are turned off. **Please double check that Sensor and Controller are both turned off and are separated when they are not in use.**

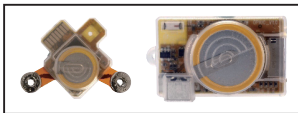

- \* Please avoid charging "over-night" from a PC, because most PC will shut down USB port after some time, and if Sensor and Controller are not properly put into "OFF" mode, their charges will drain off again after the USB is shut down.